RIDDER

Assistent

★★★★★ PREMIUM

RIDDER GmbH

Ridder Straße 1, D-56379 Singhofen
Tel.: +49 (2604) 9541-0, Fax +49 (2604) 9541-50
Mail: info@ridder-online.de, www.ridder-online.de

Art.-Nr.: A030030107

INDOOR-ROLLATOR

LIEFERUMFANG / DELIVERY CONTENT

1x Indoor Rollator
Indoor walking frame

1x Tablett
tablet

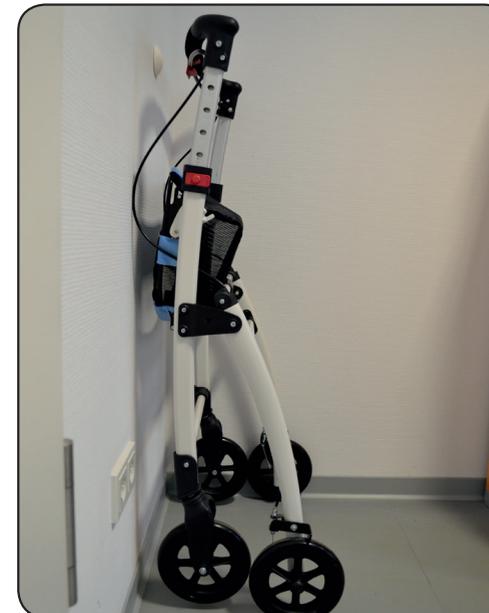
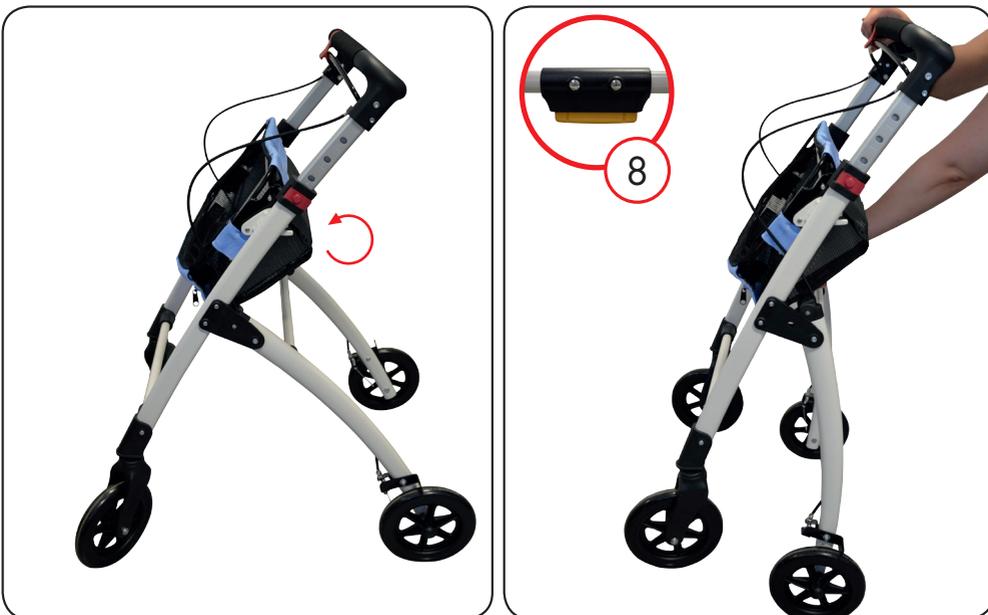
1x Korb
basket

1x Anleitung
instruction manual

ABSTELLEN / PARKING

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Instruction Indoor Rollator A0300301

Dear Customer,

Thank you for choosing our product. Please read the safety instructions carefully before use! The manufacturer assumes no liability for improper use of the Rollator /Walking frame.

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including removable basket

SAFETY SUMMARY

DO NOT install or use this equipment before reading and understanding of this instruction sheet. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment – otherwise, injury or damage may occur.

Each individual should always consult with her/his physician or therapist to determine the proper adjustment and usage.

- Only use on flat and solid ground
- DO NOT sit on the rollator/walking frame
- maximum strain: 100 kg
- maximum strain of the tablet: 4,5 kg
- maximum strain of the basket: 4,5 kg
- Always check the brakes after every height adjustment in order to avoid a free flex of the cable.
- Always check whether both of the handle tubes are secured at the same height and the snap buttons for the handgrip height adjust are fixed correctly.

Failure to follow these warnings can result in personal injury.

All wheels **HAVE TO** be in contact with the floor at **ALL TIMES** during use. This will ensure the Rollator/Walking frame is properly balanced.

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Important: Check all parts for shipping damage. In case of shipping damage, DO NOT use the product, contact the Dealer/Carrier for further instructions.

stability warnings

The Rollator/Walking frame can provide ambulatory assistance to an individual weighing up to 100 kg, INCLUDING the loading of the basket. The Rollator/Walking frame basket has a weight limitation of 4.5kg